

DR EDSW

User Guide



Attention

Respected user:

Thank you for choosing the instruments which carefully designed and manufactured by our company.

In order to better utilize the product, we sincerely recommend:

◆◆The DR EDSW is designed for personal use. Thanks to its innovative design, an individual can perform the entire treatment themselves by following the instructions in this manual.

◆◆The DR EDSW delivers shockwaves. These powerful, targeted sound waves are not quiet. The loud sound you hear when using The DR EDSW is normal and necessary to deliver clinical-strength therapy.

◆◆The DR EDSW uses an electromagnetic coil to generate intensity to propel the bullet forward. It is normal for the coil and handle to be warm.

◆◆A full treatment cycle involves treating each line of treatment twice, in order, as indicated by the smart guidance system.

◆◆The DR EDSW is not cordless/rechargeable. It must be plugged in during use.

◆◆You may notice fine black powder or dust near the tip of the device during or after use. This is the normal result of wear inside the DR EDSW due to the powerful mechanical collisions which are necessary to create the acoustic waves. This powder may be safely wiped off of the device during regular cleaning and it is not dangerous if it contacts your skin.

Notice: Please read this manual carefully before operating this machine.

DR EDSW Therapy Machine

【1】 Product Appearance (For reference only, subject to the actual product)



【2】 Standard accessories

			
Machine*1	Shockwave Handle*1	Working heads*2	Power line*1

【3】 Technical Parameters

Energy	10-35mj	Net Weight	3.8KG
Frequency	4-16Hz	Working Heads	2 Pieces
Bullet Lifespan	2million shots		
Touch Screen Size	4.3 inch		


【4】 Structure of the machine




Structure: ①Screen ②Knob ③Shockwave Outlet ④Shockwave Handle ⑤Start Button ⑥Working Heads
⑦Cooling Fan ⑧Power Switch ⑨Power Outlet

【5】 Use Process

(1) Install handles

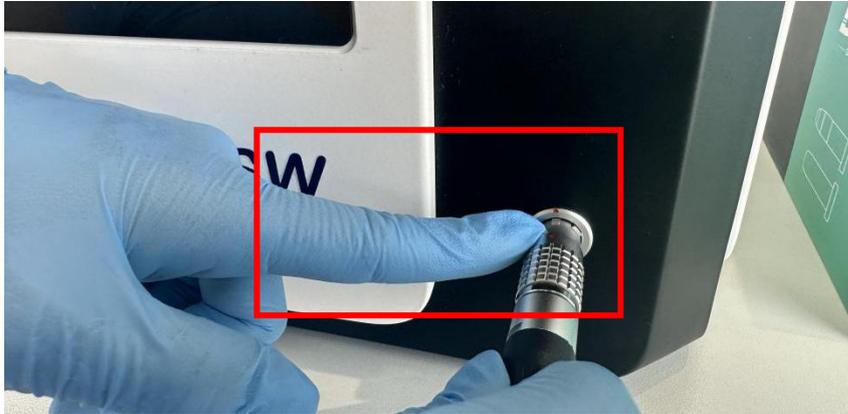


(2) Plug in and power on

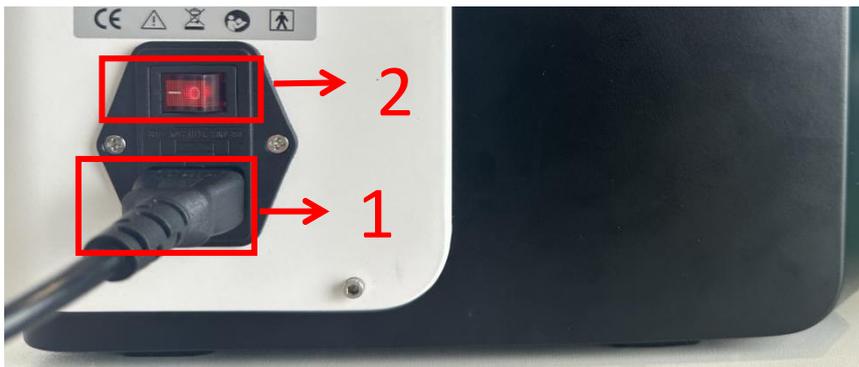


(3) Enter password: "888888"



(4) Main Interface To Adjustment Parameter

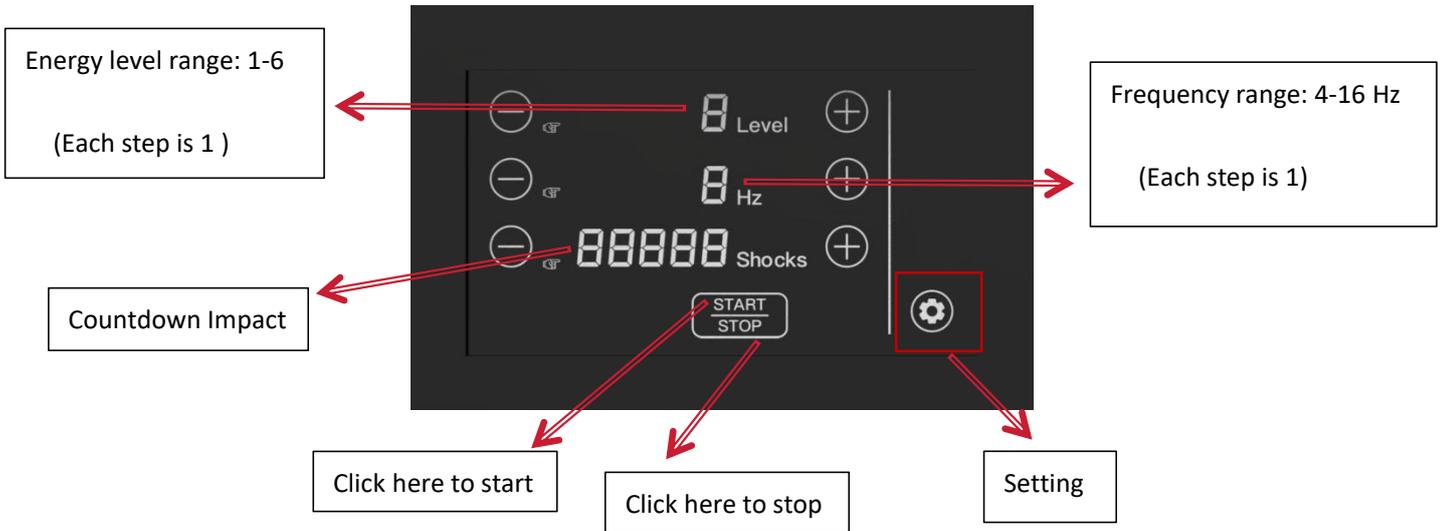

(5) Click "Setting" Enter The Usage Instructions

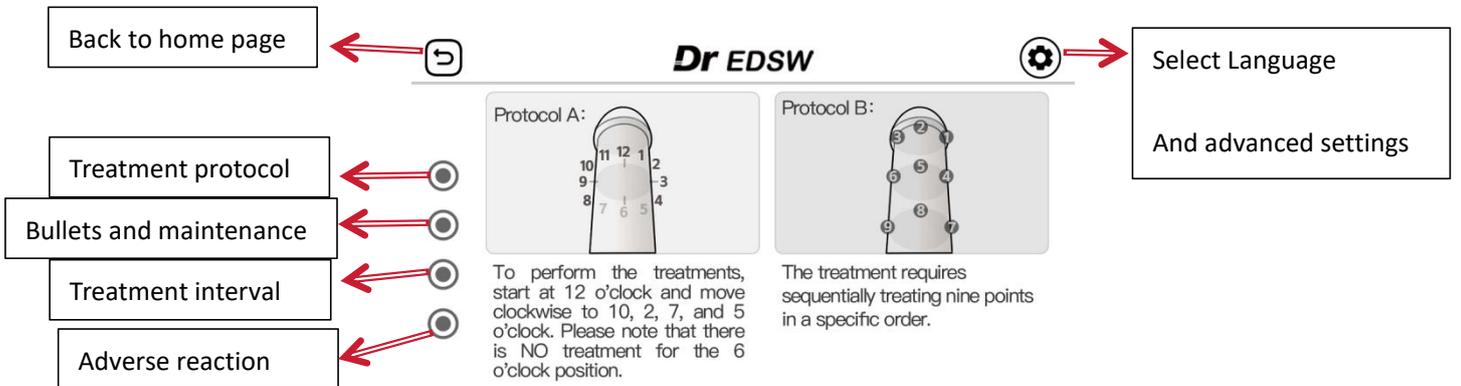

Protocol A: Linear hit along the 5 directions. From top to Base. 900 shocks for each direction.

Protocol B: Hit at 9 points in a specific order. 500 shocks for each point


(6) Treatment Recommendation

Days1-30	2 Treatments/Week	During initial treatments, tissue responds, but results may not be noticeable.
Days30-60	Recovery	Extended recovery period with no treatments.
Days60-90	2 Treatments/Week	Perform another cycle of treatments; many report improvement.
Days90-120	Recovery	Final recovery period for full tissue response.



(7) Adverse reaction

- Experiencing Discomfort During Use: Ensure the treatment tip is in contact with your shaft before starting. Apply moderate pressure for proper acoustic coupling. If discomfort persists, stop immediately.
- Bruising, Swelling, or Bleeding: Minor bruising in the treatment area may occur. Use moderate pressure to prevent it. If you experience swelling, bleeding, stop immediately.
- No Discernible Results:
 - ① Complete the full treatment protocol as it is not an overnight solution.
 - ② Contact us via chat or phone to confirm correct treatment procedures.
 - ③ Consult your physician to assess your overall health, as other factors may impede maximum results.

[6] More Photos

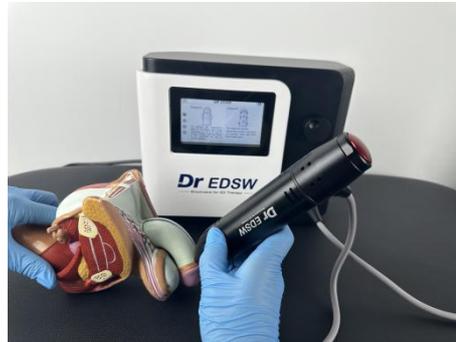

【7】 Cleaning DR EDSW After Use
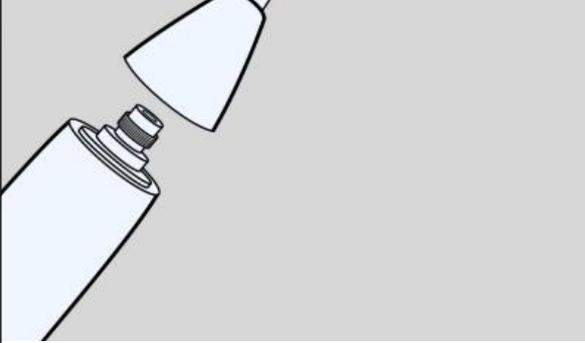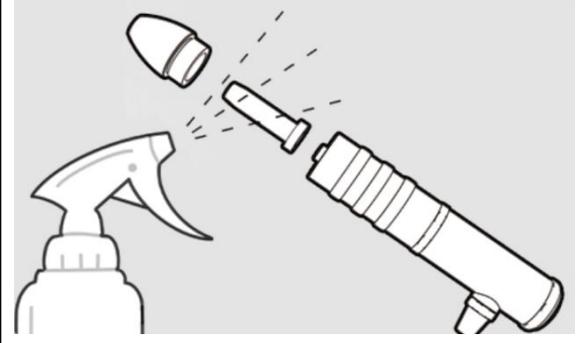
<p>①</p> 	<p>②</p> 
<p>Unplug The DR EDSW from the power adapter and unscrew the nose cone to fully remove the treatment tip.</p>	<p>Remove the accessory by un-plugging and unscrewing the nose cone.</p> <p>Clean the accessory surfaces with rubbing alcohol or soap and water.</p>

【8】 Troubleshooting Guide

Problem	Possible Solution
<p>Feeling Pain During Use</p>	<ul style="list-style-type: none"> • Make sure the treatment tip is in contact with your shaft before pressing the start button. Otherwise, initial contact could be uncomfortable. • Reduce the pressure applied during treatment to a more comfortable level. Moderate pressure is required for appropriate acoustic coupling. • If pain continues, stop treatments and consult your physician.
<p>Bruising, Swelling, or Bleeding</p>	<ul style="list-style-type: none"> • Minor bruising in the treatment area is possible for some men. This may be avoided by using moderate pressure in the treatment area. • Stop immediately if you experience swelling, bleeding, or prolonged numbness.

<p>I Have Not Noticed Any Results</p>	<ul style="list-style-type: none">• The DR EDSW is not an overnight fix.• Contact us via chat or phone to confirm that treatments are being done properly - we want to do everything we can to help you experience the same success thousands of other men are enjoying.• Consult your physician to confirm that the rest of your health picture is in good shape. It's possible something else is holding you back from maximum results.
---------------------------------------	---

DR EDSW

-----Embrace Your Sexual Well-being

